

Anti-Human CD117/c-Kit/SCF [Polyclonal]

Catalog No.	Description
AR759-5R	6 ml of Ready-to-Use Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system
AR759-10R	10 ml of Ready-to-Use Antibody in a barcode labeled vial for use with BioGenex Super Sensitive™ Detection Systems and i6000™ Automated Staining Systems
PU759-UP	1 ml of Concentrated Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system
PU759-5UP	0.5 ml of Concentrated Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system
AW759-YCD	Ready-to-Use Antibody in Barcode labeled vial for use on the Xmatrx® Elite/Ultra Staining System, 160 tests
AW759-50D	Ready-to-Use Antibody in Barcode labeled vial for use on the Xmatrx® Elite/Ultra Staining System, 50 tests

Clone	Species	Ig Class
Polyclonal	Rabbit	IgG

Intended Use

For In Vitro Diagnostic Use. This antibody is designed for the specific localization of CD117/c-Kit/SCF in formalin-fixed, paraffin-embedded (FFPE) tissue sections. Evaluation must be performed by a qualified pathologist.

Summary and Explanation

CD117 is a cytokine receptor expressed on the surface of hematopoietic stem cells as well as other cell types. CD117 recognizes a protein of 145kDa, which is identified as CD117/p145kit. This rabbit polyclonal antibody does not interfere with the binding of the stem cell factor (SCF) to c-kit, and it precipitates both the unoccupied as well as the occupied form of c-kit. The binding of SCF to the c-kit-encoded receptor tyrosine kinase (Type III) stimulates a variety of biochemical responses that culminate in cellular proliferation, migration, or survival. C-kit plays an important role in hematopoiesis, melanogenesis, and gametogenesis.

Storage and Handling

Store at 2-8°C. Fresh dilutions, if required, should be prepared prior to use and are stable and steady for up to one day at room temperature (20-26°C). Diluted antibody preparations can be refrigerated or frozen for extended shelf life.

Principles of the Procedure

Antigen detection by immunohistochemistry (IHC) is a two-step process wherein the primary antibody binds to the antigen of interest and that binding is detected by a chromogen. The [primary antibody](#) may be used in IHC using manual techniques or BioGenex Automated Staining System. Positive and negative controls should always be run simultaneously with all patient specimens.

Reagents Provided

Rabbit Polyclonal Antibody CD117/c-Kit/SCF is affinity purified and diluted in PBS, pH 7.2, containing 1% BSA and 0.09% sodium azide.

Dilution of Primary Antibody

BioGenex Ready-to-Use antibodies have been optimized for use with the recommended BioGenex Detection System and should not require further dilution.

BioGenex concentrated antibodies must be diluted in accordance with the recommended protocol when used with the recommended BioGenex Detection System.

Recommended Protocol

Refer to the following table for conditions specifically recommended for this antibody. Refer to the BioGenex website for guidance on specific staining protocols or other requirements.

Parameter	BioGenex Recommendations
Control Tissue	GIST as available with Biogenex FB-759P* & FG-759P*
Recommended Dilution for Concentrated Antibody	1:10-25 in HK941
Recommended Pretreatment (Manual/i6000)**	EZ-AR2 (HK522-XAK)
Recommended Pretreatment (Xmatrx)	EZ-AR2 Elegance (HX032-YCD)
Antibody Incubation (Manual/i6000)	30-60 min at RT
Antibody Incubation (Xmatrx)	30-60 min at 25°C
Detection System for Manual, Xmatrx & i6000 systems***	Use BioGenex Two-Step OR One-Step Super Sensitive™ Polymer-HRP IHC Detection System/DAB; see p. 2 for more information

*FB: positive control micro chamber slides, FG: positive control microscopic slides. Xmatrx requires micro chamber slides.

**Pretreatment times will vary based on individual microwave power.

***For automation systems (Xmatrx-Elite, Xmatrx-Ultra & i6000 Diagnostics), refer to the factory protocols provided with the instrument.

Category	Antibodies	Revision No.	J
Document No.	932-759P-EN	Release Date	12-Nov-2021

Detection System	Two-Step HRP Kit	One-Step HRP Kit	Link and Label Kit
Manual	QD440-XAKE (1000 Test)	QD630-XAKE (1000 Test)	QP300-XAKE (1000 Test)
	QD430-XAKE (1000 Test)		
	QD420-YIKE (500 Test)	QD620-XAKE (500 Test)	QP900-9LE (500 Test)
	QD400-60KE (60 Test)		
Xmatrix - Automation	QD550-YCDE (200 Test)	QD610-YADE (200 Test)	N/A
i6000 - Automation	QD410-YAXE (200 Test)	QD610-YAXE (200 Test)	N/A
For more information, visit www.biogenex.com .			

Precautions

This product contains sodium azide at concentrations of less than 0.1%. Sodium azide is not classified as a hazardous chemical at the product concentrations, but proper handling protocols should be observed. For more information, a Safety Data Sheet (SDS) for sodium azide is available upon request. Dispose of unused reagents according to Local, State and Federal Regulations. Wear suitable Personal Protective Equipment, do not pipette reagents by mouth, and avoid contact of reagents and specimens with skin and mucous membranes. If reagents or specimens come in contact with sensitive area, wash with copious amounts of water.

Quality Control

Refer to BioGenex detection system documents for guidance on general quality control procedures.

Troubleshooting

Refer to the troubleshooting section in the documentation for BioGenex Detection Systems (or equivalent detection systems) for remedial actions on detection system related issues, or contact BioGenex Technical Support Department at 1-800-421-4149 or support@biogenex.com or your local distributor to report unusual staining.

Expected Results

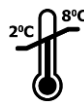







This antibody stains membrane/cytoplasm in positive cells in formalin-fixed, paraffin embedded tissue sections. An example image of a tissue section stained with this antibody can be found on the product page on the BioGenex website. Interpretation of the staining result is solely the responsibility of the user. Experimental results should be confirmed by a medically-established diagnostic product or procedure.

Limitations of the Procedure

Improper tissue handling and processing prior to immunostaining can lead to inconsistent results. Variations in embedding and fixation or the nature of the tissue may lead to variations in results. Endogenous peroxidase activity or pseudo peroxidase activity in erythrocytes and tissue biotin may result in non-specific staining based on the detection system employed. Tissues containing Hepatitis B Surface Antigen (HBsAg) may give false positive with horseradish peroxidase systems. Improper counterstaining and mounting may compromise the interpretation of results.

Bibliography

- Yarden Y, et al: "Human proto-oncogene c-kit: a new cell surface receptor tyrosine kinase for an unidentified ligand". *EMBO J*.6 (11): 3341–51
- Edling CE, et al. "c-Kit--a hematopoietic cell essential receptor tyrosine kinase". *Int. J. Biochem. Cell Biol.*39 (11): 1995–8.
- Kitamura Y, et al: "Kit as a human oncogenic tyrosine kinase". *Cell Mol Life Sci*. 2004;61:2924–2931.
- Chabot B, et al. "The proto-oncogene ckit encoding a transmembrane tyrosine kinase receptor maps to the mouse *W* locus". *Nature*. 1988;335:88–89.
- Zsebo KM, et al. "Stem cell factor is encoded at the *S1* locus of the mouse and is the ligand for the c-kit tyrosine kinase receptor". *Cell*. 1990;63:213–224.
- Center for Disease Control. Decontamination of Laboratory Sink Drains to Remove Azide Salts. Center for Disease Control Manual Guide--Safety Management, No. CDC-22, Atlanta, Georgia. April 30, 1976.
- Nadji M, Morales AR. Immunoperoxidase, part 1: the techniques and its pitfall. *Lab Med* 1983; 14:767-770.
- Omata M, Liew CT, Ashcavai M, Peters RI. Nonimmunologic binding of horseradish peroxidase to hepatitis B surface antigen. A possible source of error in immunohistochemistry. *Am J ClinPathol*. May, 1980;73(5):626-632.
- U.S. Congress. Clinical Laboratory Improvement Amendments of 1988: Final Rule, 57 FR 7163, February 28, 1992.
- National Institute for Occupational Safety and Health, (NIOSH), Rockville, MD. *Explosive azide hazard*, Publication No. 78-127, 1976.

	Temperature Limitation		In Vitro Diagnostic Medical Device
	Use By Date		Batch Code
	Non-Sterile		Consult Instructions for Use
	Representative in the European Community		Manufacturer

© 2020, BioGenex Laboratories. All rights reserved.

Category	Antibodies	Revision No.	J
Document No.	932-759P-EN	Release Date	12-Nov-2021