

EZ-Retriever[®] IR System

All-in-One All-at-Once

- Precise temperature detection thru IR sensor
- Large touchscreen for easy setup of protocols
 - Dewax, rehydration, and antigen retrieval
 - 96 slides in under 30 min
 - Superheating fluid: No boils and spills
 - Ultraclean intense stain
 - Antigen, nucleic acid and decal retrieval



DEFAULT Protocols	
RUN1	RUN2
95 °C 5MIN	95 °C 5MIN
95 °C 5MIN	107 °C 5MIN
60 °C 5MIN	90 °C 5MIN
CONTINUE	
CANCEL	

SUPERIOR QUALITY


REAL FAST REAL EZ

Antigen Retrieval, invented by BioGenex, is an effective way of unmasking antigenic epitopes on the surface of formalin-fixed, paraffin-embedded tissue sections using microwave heating. The Antigen Retrieval technique breaks the formalin-induced cross-linking bonds between epitopes and unrelated proteins, allowing better penetration of antibody and accessibility of epitopes. Covered by exclusive patents issued to BioGenex, this method has been practiced in laboratories throughout the world.


Product	Xmatrx® Elite 200 Slides (RTU) *	Xmatrx® Infinity 200 Slides (RTU)*	Manual 1000 mL (RTU)
EZ-AR™ 1 Elegance	HX031-YCD	HX031-YCX	HK546-XAK
EZ-AR™ 2 Elegance	HX032-YCD	HX032-YCX	HK547-XAK

*80 µL/test for Xmatrx Elite, 70 µL/test for Xmatrx Infinity


Results with EZ-AR™ Elegance series

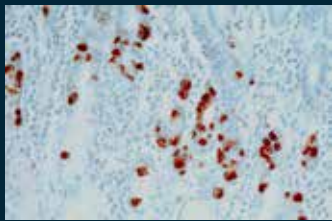


Gastrin on Stomach

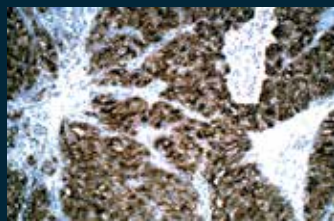


RCC-26

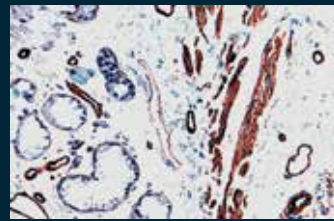


Laminin on Lung

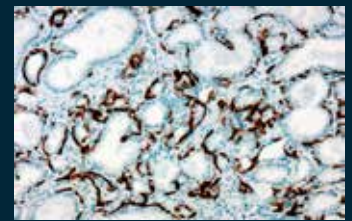


CK5+14 on Prostate

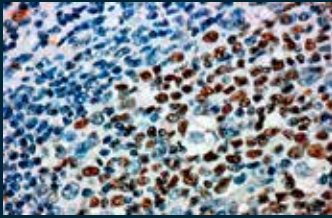


BCL6 on Tonsil

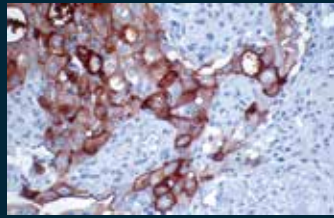

CA125 on Ovarian Carcinoma

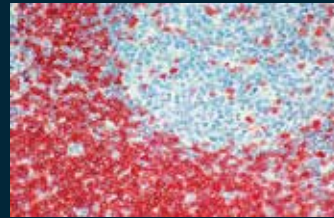

CD5 on Lymph Node

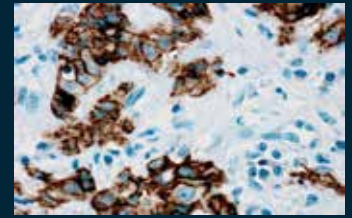

E-Cadherin on Breast Carcinoma

Parameters	EZ-RT (MW015-IR)	EZ-RT (MW016-IR)
Slide capacity	96 Slides	96 Slides
Oven capacity	30L	34L
Operating voltage	230 V AC, 50Hz	110 V AC, 60Hz
Microwave power	1500 W	1550 W
Dimensions (W/D/H)	24" X 16" X 12"	23" X 18" X 13"
Weight	46.3 lbs / 21kg	46.3 lbs / 21kg

Contact Information:

USA: 48810 Kato Road, Suite 200E, Fremont, CA 94538, USA

India: APIIC SEZ, Adibatla, R.R.Dist. 501510, Telangana, India

China: Unit 511B, 5F Front Hall, Shanghai Exhibition Center, Shanghai 200041, PRC, China

Customer service - US:

Customer service - India:

Customer service - Global:

Instrument support:

Reagent support :

customerservice@biogenex.com

indiacs@biogenex.com

internationalcs@biogenex.com

instrument.care@biogenex.com

reagentsupport@biogenex.com

For US

T: +1-510-824-1400

T: +1-800-421-4149

F: +1-510-824-1490

For India

T: +91-6309448856

For China

T: +86-21-2220-6680

F: +86-21-2220-6679

For outside US, India, and China

T: +91-7702301043



www.biogenex.com