



Antigen Retrieval Solutions

BioGenex is the pioneer of developing antigen retrieval solutions, decal retrieval in decalcified tissues, and nucleic acid retrieval for immunohistochemistry (IHC), and in situ hybridization (ISH) applications. BioGenex's new EZ-AR™ Elegance antigen retrieval solutions eliminate common issues such as improper heat distribution, buffer evaporation, tissue drying, and boil-overs during antigen retrieval. Using EZ-AR™ Elegance solutions with the EZ-Retriever® Microwave System streamlines the retrieval processes, including Deparaffinization, rehydration, and antigen retrieval.

Antigen retrieval is an effective method of unmasking antigenic epitopes on the surface of formalin-fixed paraffin-embedded (FFPE) tissue sections. The antigen retrieval technique breaks the methylene bridges between epitopes and unrelated proteins to expose antigenic sites for antibody binding. Covered by exclusive patents issued to BioGenex, this method has been practiced in laboratories throughout the

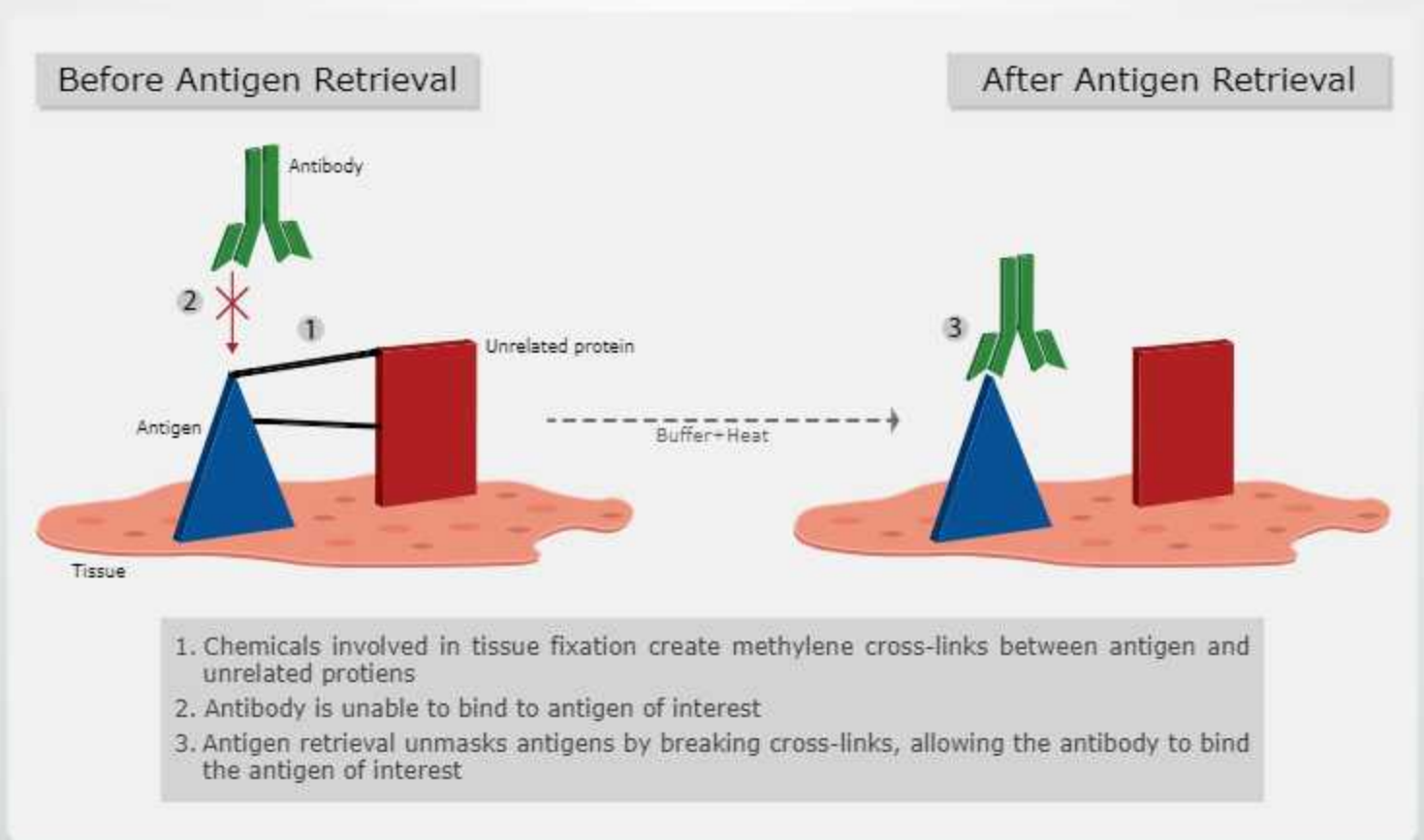


Results with EZ-AR™ Elegance series

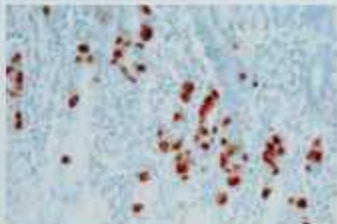


Gastrin on Stomach



RCC-26

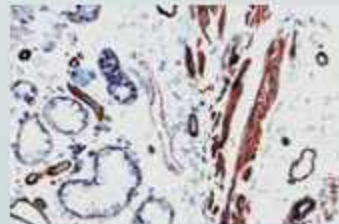

Laminin on Lung

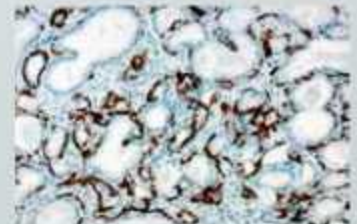

CK5+14 on Prostate

Products	100ml (10x)	250ml (RTU)	500ml (10x)	1000ml (RTU)	Xmartx® Elite 200 tests	Xmartx® Infinity 200 tests
Antigen Retrieval AR-10 (pH-10 @ 95°C temp)	HK057-5K	HK058-5K	-	HK058-20K	-	-
Antigen Retrieval Citra (pH-6.0 @ 95°C temp)	HK086-5K	HK087-5K	HK086-9K	HK087-20K	-	-
Antigen Retrieval Citra Plus (pH-6.2 @ 95°C temp)	HK080-5K	HK081-5K	HK080-9K	HK081-20K	-	-
Antigen Retrieval solution EDTA (pH-9 @ 95°C temp)	-	-	-	HK549-XAK	-	-
DeCal Retrieval Solution	-	HK089-5K	-	-	-	-
HSE Retrieval Solution	HK169-5K	-	-	-	-	-
EZ-AR COMMON (@ 95°C - 98°C temp)	-	-	-	HK545-XDK	-	-
EZ-AR™ 1 (pH-6.0 @ 95°C - 105°C temp)	-	-	-	HK521-XAK	-	-
EZ-AR™ 1 Elegance (pH-6.0 @ 95°C - 105°C temp)	-	-	-	HK546-XAK	HX031-YCD	HX031-YCX
EZ-AR™ 2 (pH-9 @ 95°C - 105°C temp)	-	-	-	HK522-XAK	-	-
EZ-AR™ 2 Elegance (pH-9 @ 95°C - 105°C temp)	-	-	-	HK547-XAK	HX032-YCD	HX032-YCX

* 2 Gallons Pack Size: HK521-XIK - EZ-AR™ 1 | HK522-XIK - EZ-AR™ 2

EZ-AR™ Elegance solutions with the EZ-Retriever® Microwave System streamline the retrieval processes, including deparaffinization, rehydration, and antigen retrieval.

Parameters	EZ-Retriever
Slide capacity	96 Slides
Oven capacity	27 L
Operating voltage	115 V AC, 60Hz
Microwave power	1100 W
Dimensions (D/W/H)	16.3" x 20.5" x 14.2"
Weight	37.5 lb/17 Kg



EZ-Retriever® System

Contact Information:

USA: 48810 Kato Road, Suite 100E & 200E,
Fremont, CA 94538, USA

India: APIIC SEZ, Adibatla, R.R.Dist. 501510, Telangana, India

China: Unit 511B, 5F Front Hall, Shanghai Exhibition Center,
Shanghai 200041, PRC, China

Customer service - US: customerservice@biogenex.com
 Customer service - India: indiacs@biogenex.com
 Customer service - Global: internationalcs@biogenex.com
 Instrument support: instrument.care@biogenex.com
 Reagent support: reagentsupport@biogenex.com

For US
 T: +1-510-824-1400
 T: +1-800-421-4149
 F: +1-510-824-1490

For India
 T: +91-9100032862
 T: +91-6309448856

For China
 T: +86-21-2220-6680
 F: +86-21-2220-6679

For outside US, India, and China
 T: +91-7702301043



www.biogenex.com

